
# L 151-2

Bedienungsanleitung  
Instructions for use  
Notice d'emploi  
Istruzioni per l'uso  
Instrucciones para el uso  
Gebruiksaanwijzing

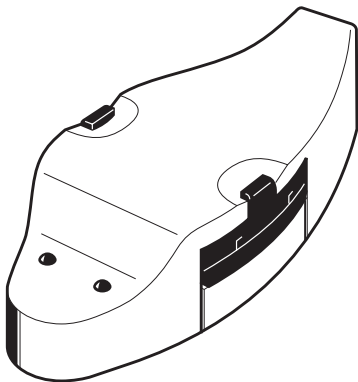# L 151-2 Ladegerät

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# L 151-2 charger

## Operating instructions

The L 151-2 charger ① can charge up to two BA 151 accupacks ②. The normal charging time is 14 hours. We strongly recommend, however, to charge new or completely discharged BA 151 accupacks for 24 hours.

When an accupack is inserted into the charger, the corresponding LED ③ lights up, indicating that the accupack is being charged.

The L 151 charger is powered via the supplied plug-in mains unit. Connect the plug-in mains unit to socket ④ on the charger. Use the charger and the plug-in mains unit in dry rooms only.

Since the accupacks cannot be overcharged, they can remain in the charger even when they have been fully charged. However, when you are not going to use the charger for some time (e.g. when you are on holiday), you should unplug the mains unit and take the accupacks out of the charger.

## Attention!

Some furniture surfaces have been treated with varnish, polish or synthetics which might cause stains when they come into contact with other synthetics. Despite a thorough testing of the synthetics used by us, we cannot rule out the possibility of discolouration, since we don't know your furniture. You should therefore always place the charger on a non-slip pad.

Use a damp cloth for cleaning the charger. Always unplug the mains unit when you wish to clean the charger. Do not use any solvents.

## Technical Data

|                                  |                                                                                   |
|----------------------------------|-----------------------------------------------------------------------------------|
| Charging compartments            | 2 compartments for BA 151                                                         |
| Charging current per compartment | approx. 6 mA                                                                      |
| Charging time                    | 14 h, initial charging 24 h                                                       |
| Dimensions (charger only)        | 50 x 100 x 30 mm                                                                  |
| Weight (without mains unit)      | approx. 50 g                                                                      |
| Delivery includes                | 1 L 151-2 charger<br>1 plug-in mains unit,<br>for 230, 120 or 240 V mains voltage |

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Sennheiser electronic GmbH & Co. KG